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Alum Stop

Aluminum Chloride %25

Colorless Solution for bleeding control and sulcular fluid.

Composition: Aluminum chloride, Hydroglycolic base
Intended use:

- ▶ Perfect transparency for better performance without staining or staining.
- ▶ Ideal viscosity.
- ▶ Without interacting with bonding agent.
- ▶ Control of bleeding and gum secretions during gingival close restorations.
- ▶ Gum Solecules Control During Molding.
- ▶ Stop the internal oral surface bleeding with a special application for the front teeth.
- ▶ Transparent Solution for anterior restorations.
- ▶ Suoclaric fluid control for better bonding in adhering indirect restorations, Laminate, veneer, Inlay, Onlay.

Advantages:

- ▶ Perfect transparency for better performance without leaving stains
- ▶ Suitable viscosity
- ▶ No interaction with the bonding agent

Precautions:

- ▶ Carefully read and understand instructions before using.
- ▶ It's necessary to use rubber-dam while working with Alum Stop
- ▶ Avoid contact with oral soft tissue, eyes or skin
- ▶ If accidental contact with soft tissues, eyes or skin occurs, immediately flush with plenty of water
- ▶ In case of consumption drink copious amount of water.
- ▶ Do not induce vomiting. Seek medical advice
- ▶ Do not inject this product
- ▶ For dental use only.
- ▶ Do not use in similar cases
- ▶ If you use the 7th bond (self-etching bonds), before using the bond, you should clean the surface of the tooth with Pumice paste and then bond. In the case of

the 5th generation bond and the use of acid etching does not require this.

- ▶ The product may irritate the mucous membrane of the mouth. Rinse immediately with water in the ringer.
- ▶ In patients with an allergy to Aluminum chloride or chemicals are not used.
- ▶ Before applying any other chemicals, the tissue should be thoroughly washed with water.
- ▶ At the end of the work, the remainder of the material must be completely removed from the patient's mouth.

warning:


- ▶ Do not use after expiration date.
- ▶ Keep out of reach of children.
- ▶ Protect from direct sunlight and Humidity.
- ▶ Store in the original container at room temperature.

Direction for Use:

- ▶ Prepare the desired surface by washing with water and mildly drying by air pooper.
- ▶ A sufficient amount of Solution on the bleeding tissue for 1-3 minutes (to prevent infection, it is better to transfer some Solution with a disposable brush or cotton swab to the desired place).
- ▶ The surface of the Solution is soaked so that the Solution is completely cleaned, washed, then the texture is completely dry.
- ▶ Air pointer should be used mildly.

Contraindication:

- ▶ This product is not contraindicated for children, pregnant women, the elderly and other specific diseases and can be used for all people who are not allergic to the active substance.

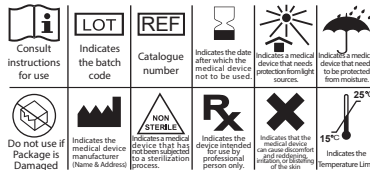Alum Stop

Aluminum Chloride %25 Solution

- ISO13485 : 2016 , US cert, IAS
- Iso 9001: 2015, Tuv certification
- Iso 14001: 2015, Tuv certification
- Iso 18001: 2007, Tuv certification
- Iso 10002: 2014, Tuv certification
- Iso 10004: 2012, Tuv certification

MEDICAL
EQUIPMENT
DEPARTMENT
IFDA
**Production
License No.
89586041
IMD: 91246235**

manufacturer
Pardis Morvabon Pharmed Co.:
Head Office: Unit 154, 7th floor, 7 block,
Bohsaz Bd, Azadi Street, Tehran, Province, Iran
Tel: 0098 21 6658 1438
Fax: 0098 21 6690 8819
Factory: Unit 135, 136, 139, 2nd floor, Seraj Bd.,
20thkm of Damavand Rd., Tehran, Iran
Tel: 0098 21 7625 1702
Fax: 0098 21 7648 1457